

 **SOULEVATI**

TOILET LIFT SEAT





SOLLiEVATI is a toilet lift designed for elderly people and for anyone suffering from limited mobility due to diseases or injuries. It can help eliminate some of the stress and strain on the joints when the user goes from sitting to standing.

The user, standing in upright position and simply leaning on the toilet seat, is gently lowered and then raised, effortlessly, safely, independently, while always keeping his feet on the ground.

SOLLiEVATI can really help these people to retain their autonomy in the bathroom safely and comfortably.

It reduces or even eliminates the need to be assisted by auxiliary personnel.



MAIN FEATURES

+ ADJUSTABLE: The seat can be stopped in any position you want between the minimum and the maximum height allowed. The arms (standard supplied) can be reclined backwards to allow the user more freedom to move.

+ ANATOMIC: SOLLiEVATI moves up and down gently and slowly reproducing the natural movement of our body.

+ NOISELESS: Thanks to two lifting silent actuators, self-lubricating axle-bearings, sound-proofing carter coating, the SOLLiEVATI lift offers noiseless operation.



+ CLEAN: Particular attention has been paid to the design of SOLLiEVATI. It is not only very nice but it is also very easy to keep clean, thanks to its smooth, clear surfaces. In addition, the coating is painted with special high-temperature scratchproof paint.

+ EASY: No need of masonry work, installed in just 1 hour. Fits almost any type of toilet.

+ COMPLETE: Arms (standard supply) are ergonomically shaped to guarantee the highest comfort and confidence to the user.

+ USER FRIENDLY: Up-down controls consist of two light buttons that can be easily operated while still holding the arm. Up-down control buttons can be on either the right or the left arm, according to user's request.

+ SAFE: On the arm supplied with up-down control buttons, you can also find the on-off button. Also this button is light to guarantee the highest safety.



SOLLiEVATI has been designed and manufactured by a company active for 40 years in the production of electromedical devices. It is an Italian patent, creation and production.

SOLLiEVATI is CE marked as a Medical Devices, accordingly to the latest MDR 2017/745 regulation.

THE SOLLiEVATI TOILET LIFT IS AVAILABLE IN 2 DIFFERENT TYPES:



SILVER: BATTERY POWER

It works by means of a lithium-ion battery that guarantees a very long endurance (50 to 80 cycles according to user's weight); it takes about 3 hours to recharge the battery. It is supplied with two batteries thus allowing continuous operation.

The battery replacement is very easy and fast (see picture on the left); this operation only takes a few seconds.

.....

GOLD: ELECTRIC WIRE

Supplied by mains current 230 v. This type is suitable for users who do not want to worry about battery charging.

N.	ARTICLE CODE	DESCRIPTION
1	BA SLVT SILVER	SOLLiEVATI WC lift - battery version
2	BA SLVT GOLD	SOLLiEVATI WC lift - electric version

OPTIONAL for SOLLiEVATI WC lift



Feet kit

Extension feet kit, to increase the lift height up to 8 cm. They also allow use on disabled toilets (with increased height).

Cover

Toilet seat cover.




N.	ARTICLE CODE	DESCRIPTION
3	BA SLVTKITP	Extension feet kit for SOLLiEVATI - 4 pcs
4	BA SLVTCOP	Toilet seat cover for SOLLiEVATI

TECHNICAL SPECIFICATION

150 kg
maximum
user weight

weight
34.4 kg

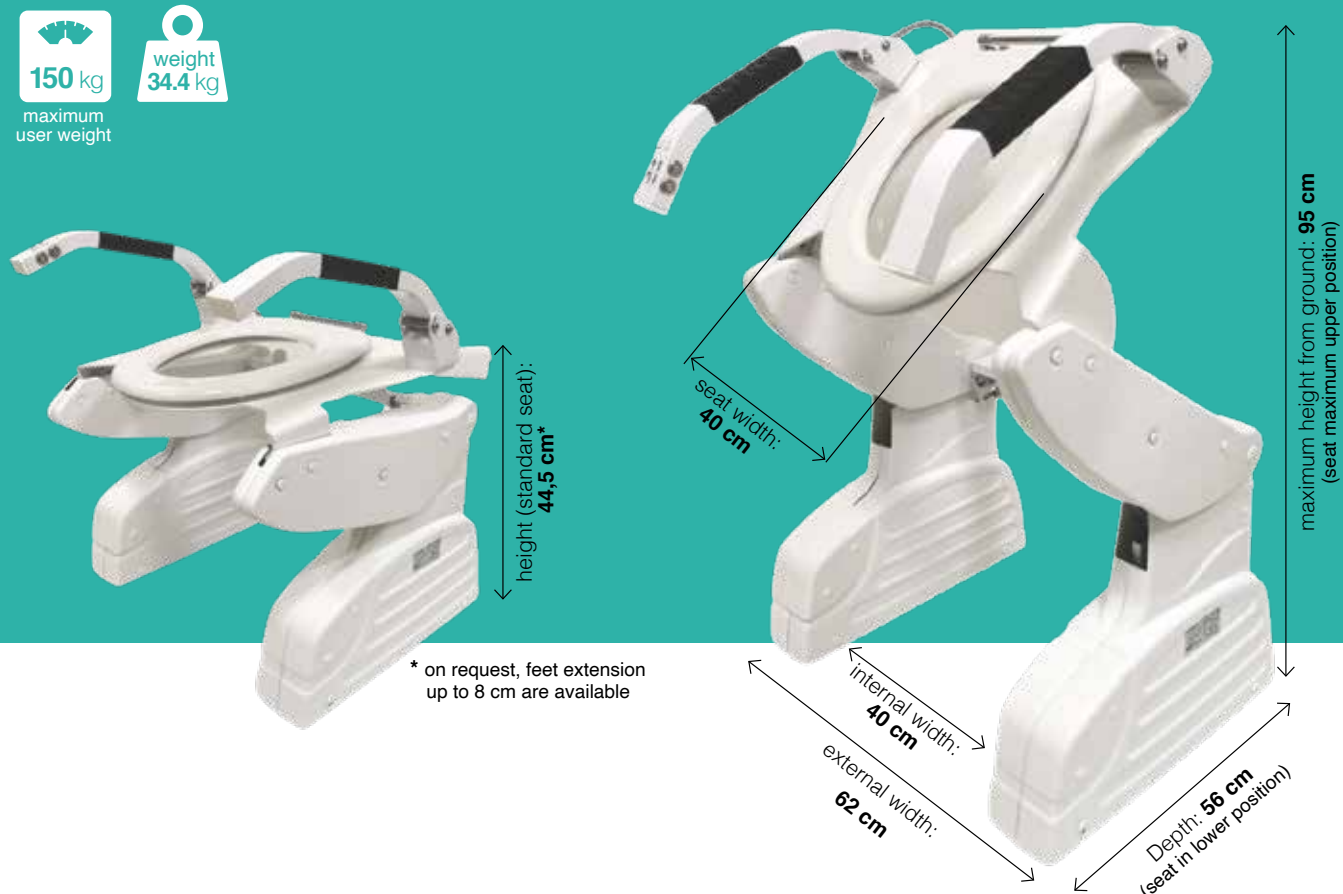

FAQ

- **What happens if the battery runs out?**

Sollievati comes with two batteries, so while one works, the other can be recharged.

- **How do I know when the battery is running low?**

When the battery is at 20% remaining charge, the LED on the armrest starts flashing.

- **What maintenance does this aid require?**

Sollievati is designed to be easily cleaned and should be cleaned like any other sanitary ware in the bathroom. Do not use aggressive, abrasive products and solvents such as bleach and hydrochloric acid to clean the lift. Do not use alcohol to clean the seat. Clean and disinfect only by wiping and not by spraying or using steam. Wipe clean surfaces with a soft cloth. The frequency of cleaning depends on the frequency of use.

- **What heights must WCs have for the use of SOLLIEVATI?**

The height of the toilet is between 38 and 44,5 cm. If the toilet is higher than 44,5 cm it is possible to install the device using the Kit feet (optional) for the extension of the height of the device.

- **How quickly do the batteries recharge? How many lifts can you do with a charged battery?**

Approximately 3.5 hours using the included power supply. About 30-50 lifts.

- **How do I know if it fits on my WC?**

Just look at the sheet illustrating the technical characteristics required to position SOLLIEVATI.